




SugarSight

www.sugarsight.tech

Install and Secure APK

Safe APK setup and updates
Step-by-step guide to install and update the
APK safely.

www.Sugarsight.tech

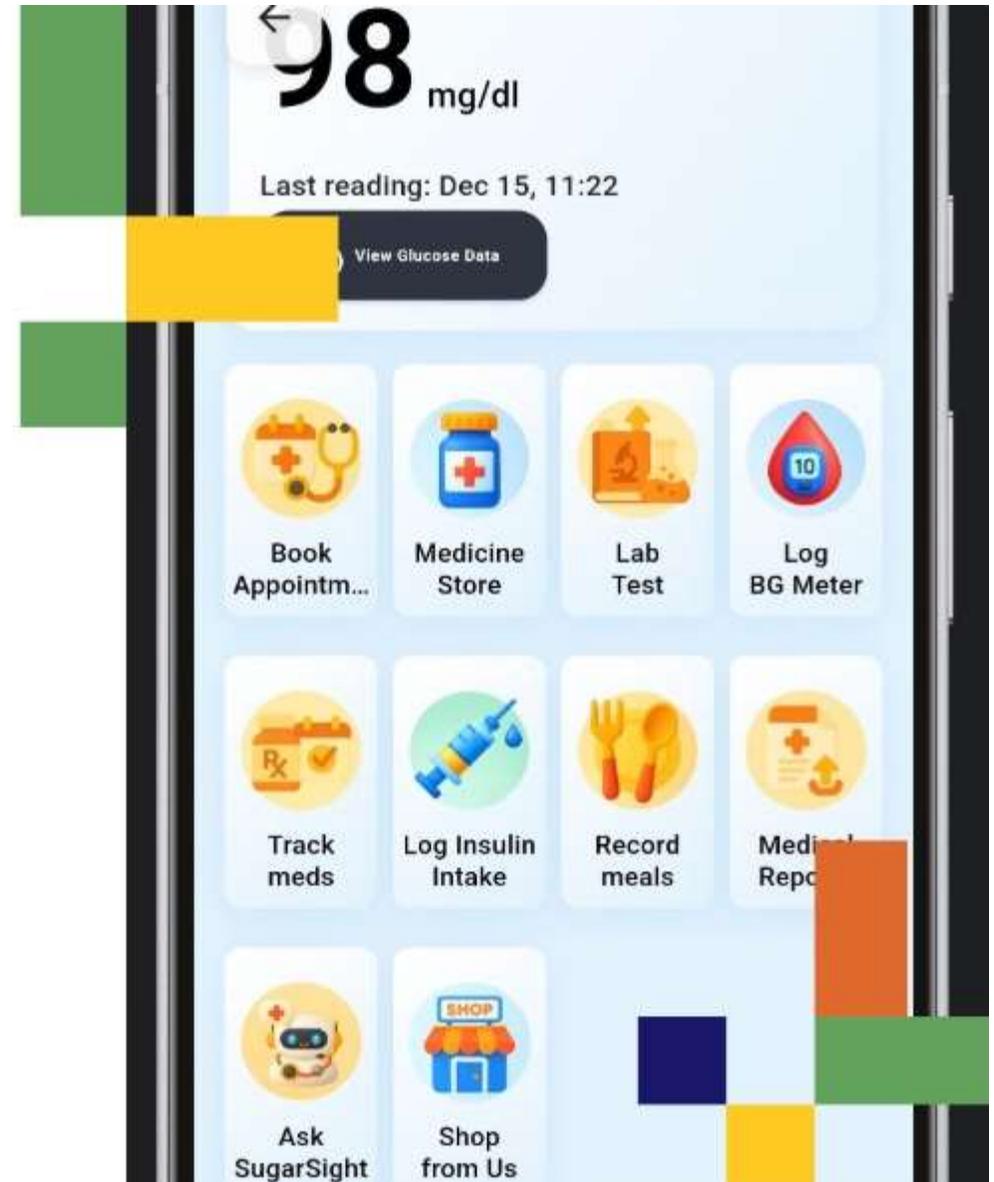

Install the APK

Quick steps to install the latest app version



Download Latest APK From Website/Email

Get The Official APK File



Open The APK File

Locate The Downloaded File And Tap To Open It.



Click Install When Prompted

Confirm The Installation By Selecting Install On The Prompt.

Settings

Allow install from unknown resources




Click on settings

security - > install unknown apps




Allow any default browser/Chrome

Toggle the button and you can install the app

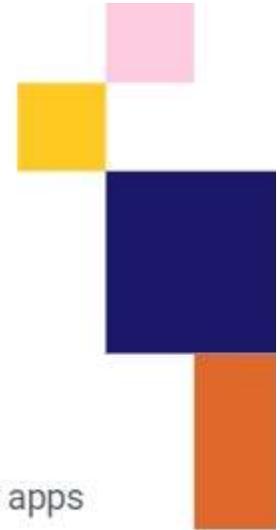

Install APK

Quick steps to install an Android APK file

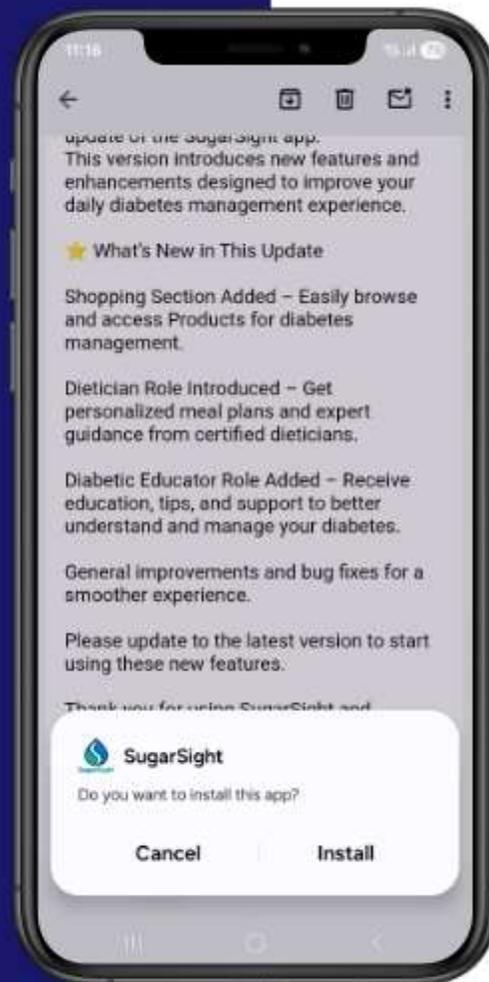

Click install

Confirm permission and tap to install the app setup



Wait for installation

Allow the installer to complete; do not close the screen until finished.

Set up APP

Quick steps to setup App



App icon appears on home screen




Open the app and complete setup

1 2 3 4 5 6 7 8 9 10 11 12

Sign up

Quick steps to Sign Up

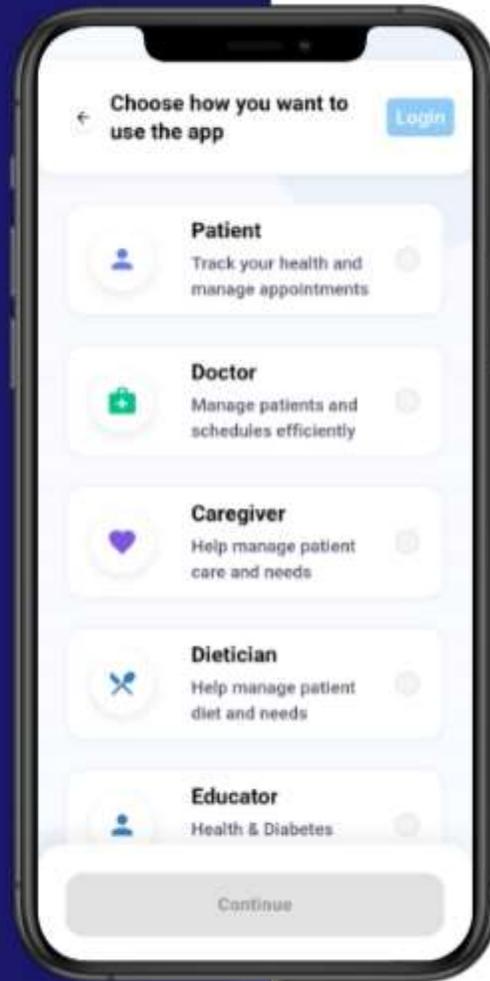

Sign Up as
Patient/Doctor/Caregiver/
Dietician/Educator



One account. Multiple roles
supported.



Update the APK

Quick steps to update the latest app version



Download Latest APK From Website/Email

Get The Official APK File



Open The APK File

Locate The Downloaded File And Tap To Open It.



Click Update When Prompted

Confirm The Installation By Selecting Update On The Prompt.

Update APK


Quick steps to update an Android APK file



App opens normally



Data remains safe

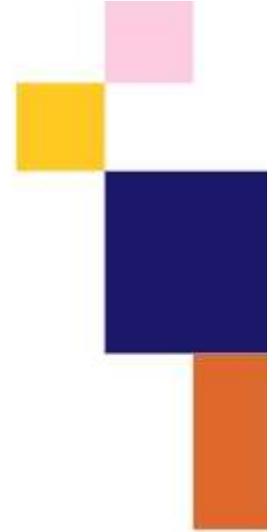

Quick Troubleshooting Steps

Fix common installation and launch problems fast



**Installation
blocked – enable
permissions**



**App not opening –
uninstall and
reinstall**

Support

Get Fast Help from Support

Contact our team or visit
help section For help.

✉ sugarsight@vtpl.tech

